



SS120V Super Shaker Bed Vibrator

(120 Volt Vibrator)

OPERATION

When activated by your Sonic Alert wake-up alarm or the signal from your Sonic Alert doorbell signaler, telephone signaler, baby cry signaler, or any clock with a standard 120V outlet the Sonic Alert Super Shaker bed vibrator Model SS120V will wake up even a heavy sleeper.

The Super Shaker bed vibrator Model SS120V uses a low voltage, UL-listed transformer to guarantee safe operation. The Super Shaker bed vibrator has strong vibrations designed especially for a long life. The Super Shaker bed vibrator also has a built-in automatic shut-off sensor so it will not overheat even if accidentally left on.

INSTALLATION

The Super Shaker bed vibrator is very safe and easy to use.

1. Place the Super Shaker bed vibrator between your mattress and box springs or under your pillow.
2. Plug the Super Shaker bed vibrator into Sonic Alerts Wake-Up Alarm, Remote Receivers or any alarm clock or timer with AC outlet.
3. When used with Sonic Alerts Wake-up alarm or an alarm clock, the Super Shaker bed will vibrate the instant your Wake-Up Alarm or clock goes off. The Super Shaker bed vibrator will continue to vibrate until the Wake-Up alarm or alarm clock is turned off.

The Sonic Alert Wake-Up Alarms, Models WA300 and USS360 have switches on the front that are used to turn on and off a lamp plugged into them. When using the Super Shaker bed vibrator with the Wake-Up Alarms, make sure to keep the switch in the "OFF" position so the Super Shaker bed vibrator is not vibrating when the clock is not ringing. Turning the switch to the "ON" position will keep the Super Shaker bed vibrator vibrating all day.

4. When using with Sonic Alert's Remote Receivers, Model SA101 or SA201, the Super Shaker bed vibrator will begin to vibrate as soon as it receives a signal from any Sonic Alert Signaler, such as the doorbell, telephone or baby cry signaler.

The Sonic Alert Remote Receiver Model SA201 has a switch on the front that is used to turn on and off a lamp plugged into it. When using the Super Shaker bed vibrator with SA201, make sure to keep the switch in the "OFF" position. Turning the switch to the "ON" position will keep the Super Shaker bed vibrator vibrating all day.

The Super Shaker bed vibrator will not overheat if you forget to turn off the alarm. If the Super Shaker bed vibrator is left on for more that approximately 10 minutes, the built-in temperature sensor will turn off the vibrator so it does not become too hot. When the Super Shaker bed vibrator cools down, the temperature sensor will automatically reset The vibrator for normal operation.